

**Spot-Check Monitor**  
**User Instruction Manual**

This manual is written for the PC-303 Spot-Check Monitor.

The manual describes, in accordance with the Spot-Check Monitor's features and requirements, the main structure, functions, specifications, correct methods for transportation, installation, usage, operation, repair, maintenance and storage, etc. as well as the safety procedures to protect both the user and equipment. Refer to the respective chapters for further details.

The manual is published in English and Creative has the ultimate right to explain the Manual.

For the user's convenience, we share the latest version analysis software of the Spot-Check Monitor on our website, go to [www.creative-sz.com](http://www.creative-sz.com) to download the latest version of the data management software.

Please contact the manufacturer or your local distributor if there are any issues downloading the software.

Version of This User Manual: Ver1.8


Product service life: 5 years (no warranty)

Issue Date: 3rd March 2016

Manufacturing date: See Label

All rights reserved

Marks in the manual:

 **Caution: Instructions must be followed to avoid causing harm to the user or patient.**

**Attention: must be followed to avoid causing damage to the Spot-Check monitor.**

 **Note: contains important information and advice about operations and application.**




## Instructions for use

Dear Customers,

Thank you for purchasing the PC-303 Spot-Check Monitor. Please read the following information before using the device.

These instructions describe the operating procedures which are to be strictly followed, read these instructions carefully before using the Spot-Check Monitor. Failure to follow these instructions can cause monitoring abnormalities, damage to the monitor and personal injury. The manufacturer is NOT responsible for the safety, reliability and performance issues or any monitoring abnormalities, personal injury and equipment damage due to user's negligence of the operation instructions. The manufacturer's warranty service does not cover such faults

### Warnings:

-  **Do NOT use the device under flammable gas conditions or in any environment that may lead to explosion.**
-  **The device and accessories should not be serviced or maintained while the device is in use.**
-  **The doctor or patient is the intended operator.**

- ❑ **Do not modify this equipment without authorization from the manufacturer.**
- ❑ **The SpO<sub>2</sub>, NIBP, Temperature, and ECG (optional) measurements are the frequently used functions.**
- ❑ **The device is IP22 and is protected against solid foreign objects of 12.5mm or greater, and protected against vertically falling water drops when the enclosure is tilted up to 15.**
- ❑ **Please check the monitor before use to verify that the accessories function safely and correctly.**
- ❑ **If the monitor is connected with other devices, the total leakage current may exceed the limitation and as a result this can cause potential danger to the user.**
- ❑ **All combinations of equipment must be in compliance with the standard of IEC 60601-1-1 medical and electric system requirements.**
- ❑ **Although biocompatibility tests have been performed on all the applied parts, under exceptional circumstances, allergic patients may have anaphylaxis. Do NOT use the monitor on patients with anaphylaxis.**
- ❑ **All connecting cables and rubber tubes of the applied parts should be kept away from the patient's neck to prevent suffocation.**
- ❑ **As a standard, please only use the components provided by the manufacturer or those that**

**are of the same model and specifications as the accessories.**

- If the monitor falls off a surface accidentally, please do NOT operate it before its safety and technical performance have been tested and positive results obtained.**
- Do NOT open the device cover without authorization. The cover should only be opened by a qualified service personnel.**
- When disposing of the monitor and its accessories, national regulations should be followed.**

## Table of contents

<b>CHAPTER 1 OVERVIEW.....</b>	<b>1</b>
1.1 Features.....	1
1.2 Product Name and Model.....	2
1.3 Intended Use.....	2
1.4 Impact on the Environment and Resources.....	3
<b>CHAPTER 2 OPERATION INSTRUCTION.....</b>	<b>3</b>
2.1 Appearance.....	3
2.1.1 The Front Panel.....	3
2.1.2 The Right and Upper Sides of The Device.....	5
2.2 Installation.....	8
2.2.1 Power Supply.....	8
2.2.2 Starting the Monitor.....	9
2.2.3 APP software download for Android Smart Phones.....	9
2.3 Taking Measurements.....	11
2.3.1 Blood Pressure Measurement.....	11
2.3.2 SpO <sub>2</sub> Measurement.....	13

2.3.3 Temperature Measurement .....	17
2.3.4 Blood Glucose Measurement (Optional) .....	19
2.3.5 ECG Measurement (Optional) .....	22
2.4 Pressure Accuracy Check Method .....	26
2.5 Symbols .....	28
<b>CHAPTER 3 MONITORING SCREEN DISPLAY .....</b>	<b>30</b>
3.1 Measuring Screen.....	30
3.2 System Setting Screen .....	32
3.3 History Data Review Screen .....	34
3.4 Data Uploading .....	35
<b>CHAPTER 4 TECHNICAL SPECIFICATIONS .....</b>	<b>36</b>
4.1 Blood Pressure Measurement.....	36
4.2 SpO <sub>2</sub> Measurement .....	36
4.3 Pulse Rate Measurement .....	37
4.4 Temperature Measurement.....	37
4.5 Blood Glucose Measurement (Optional).....	38
4.6 ECG Measurement (Optional) .....	38
4.7 Others.....	38

4.7.1 Operating Environment.....	38
4.7.2 Classification.....	39
<b>CHAPTER 5 TROUBLESHOOTING.....</b>	<b>40</b>
<b>CHAPTER 6 PACKING LIST.....</b>	<b>41</b>
<b>CHAPTER 7 MAINTENANCE AND SERVICE.....</b>	<b>42</b>
7.1 Technical Maintenances.....	42
7.1.1 Daily Examination.....	42
7.1.2 Routine Maintenance.....	42
7.1.3 Battery Maintenance.....	42
7.2 Cleaning and Disinfection of the Main Unit.....	44
7.3 Cleaning and Disinfection of the Accessories.....	45
7.4 Storage.....	46
7.5 Transportation.....	46



## CHAPTER 1 OVERVIEW

### 1.1 Features

- Small in size, light in weight, easy to carry and operate;
- Clear and large numeric display, segmented LCD panel, real-time clock display; Accurate blood pressure measurements can be activated or canceled by one shortcut button;
- Unique oximetry technique ensures quick and accurate SpO<sub>2</sub> & pulse rate measurements by smart sensors;
- Smart infrared temperature probe ensures quick and accurate measurements of body temperature;
- Blood pressure, oxygen saturation, pulse rate and temperature can be measured simultaneously;
- Blood Glucose meter option can be connected to the device ;
- Data storage with recall, up to 999 groups of records can be stored and recognised by patient ID.
- Power management with power saving mode, auto power off and low battery indicator;
- Data upload to PC by USB cable and real-time data transmission to smart phones by wireless connections.

## 1.2 Product Name and Model

Name: Spot-Check Monitor

Model and Configuration:

Model	Configuration				LCD display
	NIBP	SpO <sub>2</sub>	Pulse Rate	Temperature	
PC-301	√	--	--	--	√
PC-302	√	√	√	--	√
PC-303	√	√	√	√	√

NOTE: 1. Spot-Check Monitor can configure with ECG and blood glucose function, details see the User Manual for Easy ECG Monitor and Super Check Glucose Meter respectively.

2. "√" means function available, and "--" means function unavailable.

## 1.3 Intended Use

The Spot-Check Monitor is a device designed for measuring the user's physiological parameters, such as non-invasive blood pressure (NIBP), oxygen saturation (SpO<sub>2</sub>), pulse rate (PR), and body temperature (TEMP). Additionally, the device can take measurements from the Blood Glucose Meter function, and ECG data from the Easy ECG Monitor (both Blood Glucose Meter and Easy ECG Monitor are certified separately). This device is applicable for use in clinical institutions and has no conditions or factors of contraindication.

## 1.4 Impact on the Environment and Resources

Low

# CHAPTER 2 OPERATION INSTRUCTIONS

## 2.1 Appearance

### 2.1.1 The Front Panel

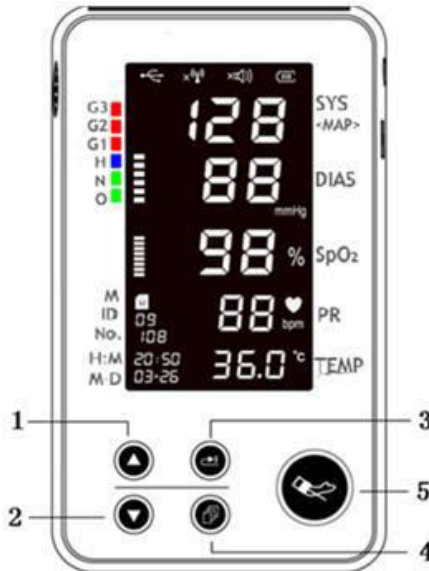









Figure 2.1


**Description:**

1/2. / up/down key: on the setup display screen, a short press will change the parameter value step by step, press and hold to change the parameter values quickly; on the review display screen, short press to review the history data records one by one, press and hold to recall the history data records quickly.

3.  Memory key: on the measurement display screen, press and hold the key (for 3 seconds) to enter into the review display screen; once the review display screen, a short press will recall the history data records.

On the setup display screen, all parameters can be set in anticlockwise order by pressing and holding the "" key, similarly, a short press of the "" key will set the parameters in clockwise order.

4.  Menu key: on the measurement display screen, press and hold the menu key to enter the setup screen; on the setup or review display screen, press and hold the "" key to go back to the measurement display screen.

5.  Start/cancel button: on the measurement display screen, a short press of this button will activate or cancel the blood pressure measurement.

## 2.1.2 The Right and Upper Sides of the Device

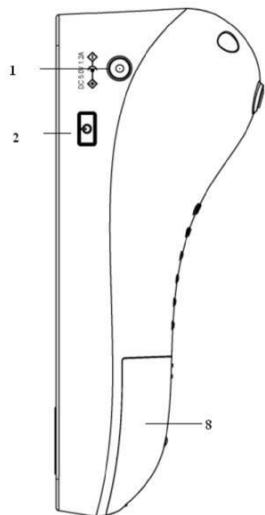


Figure 2.2


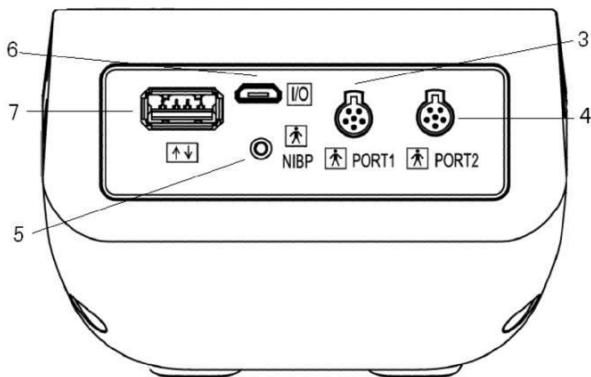

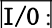
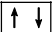


Figure 2.3A

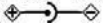

The power switch and external DC power input socket are on the right side of the monitor as shown in figure 2.2.

The signal input/output ports are on the upper side of the monitor as shown in figure 2.3A (for the previous version).

**Description:**

- DC 5.0V 1.2A
1.  : External DC power input socket.
  2.  : Power switch: = press and hold to turn the monitor on/off.
  - 3/4. **Port 1/Port 2:** Connector to link with the temperature probe or smart SpO<sub>2</sub> probe.
  5. **NIBP:** Cuff connector.
  6.  : Charge / USB data interface.
  7.  : Connector to link with the blood glucose meter.
  8. Battery cover.

NOTE: Figure 2.3A is the upper-side-view for the previous version device, and Figure 2.3B is the upper-side-view for the current version device. The difference between the two versions is seen on the

upper-side panel. The previous version device has only 2 "  " ports, marked "PORT1" and "PORT2", which are the generic connectors capable of connecting any combination of temperature probe, smart SpO<sub>2</sub> probe or ECG accessory (for example Easy ECG Monitor). However, the current version device has 3 "  " ports, marked "SpO<sub>2</sub>", "TEMP" and "ECG" respectively, which can be used only to connect the

corresponding sensors or accessories.

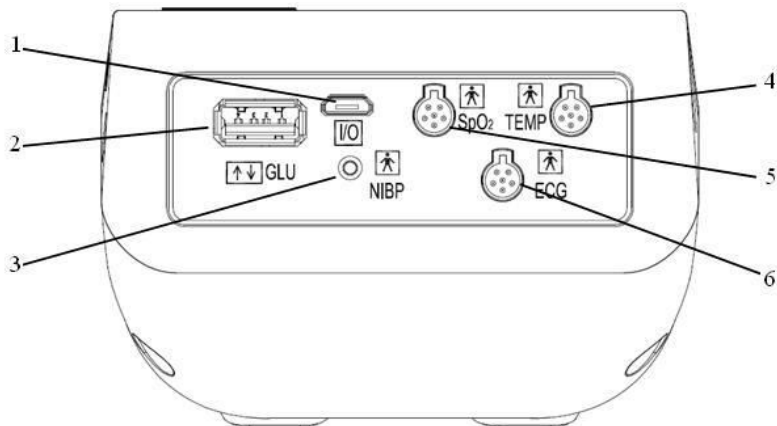
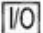



Figure 2.3B Upper-side view

**Description:**


1. : Charge / USB data interface.
2. : Connector to link to the blood glucose meter.
3. NIBP: Cuff connector.
4. TEMP: Temperature probe connector.

5. SpO<sub>2</sub>: Smart SpO<sub>2</sub> probe connector.
6. ECG: Connector to link with ECG accessories.

## 2.2 Installation

### 2.2.1 Power Supply


1. Internal power supply to the built-in battery

When the battery indicator " displays full grids, the built-in battery is fully charged. When it blinks, the battery voltage is low, and the user should charge the battery by connecting the device to the AC power adapter or a USB power source via USB cable. When the grids of the battery indicator are rolling circularly, the battery is being charged.

2. External power supply from the AC power adapter

Use the AC power adapter provided by the manufacturer. Make sure that the mains power supply is 110-240VAC with 50/60Hz.

3. External power supply from the USB cable

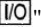Use the USB data cable with micro-USB connector, connect one end of the data cable to the connector on the device marked "" , and the other end to the USB power source with output capacity of 5Vdc/1.2A.



### **2.2.2 Starting the Monitor**

By pressing and holding down the switch, the software version will be displayed, after releasing the switch, the device will enter the measurement display screen automatically. The user can then begin to operate the monitor.

¾If the monitor fails to start by pressing the switch, please use the external power supply.


### **2.2.3 Downloading APP software onto Android smart phones**

Terminal devices such as Android smart phones can be used to receive data from the Spot-Check Monitor in real-time, store the received data, and also review the stored data. To use this function, download the corresponding APP software onto the smart phone device. Please follow the procedure to download:

1. Install an APP software for scanning QR Codes with a smart phone, such as QuickMark, I-Nigma, Bee Tagg etc..
2. Run the APP software to scan the QR Code image in Figure 2.4, please focus the QR Code frame while scanning.
3. When successfully scanned, a web link for downloading the APP software "PC-303. Apk" will be displayed.
4. Open the web link to download the APP software "PC-303.apk". Install the software when successfully downloaded.

For terminal devices with the iOS system (such as iPhone, iPad), please follow this procedure to download:

1. On the App Store of the device, enter "Shenzhen Creative" into the search function. Note: if you use an iPad to search, please select "iPhone only" when searching.

2. Once the search results are listed, select the result with @health icon "", then download the corresponding APP software.

### **Instruction for Measurement**

☞ **Make sure the APP software successfully connects with the Spot-Check Monitor.**

☞ **Refer to the manual of the APP software for more detailed information for operation.**



Figure 2.4 QR Code Image

## 2.3 Taking Measurements

### 2.3.1 Blood Pressure Measurement

1. Applying the cuff; unfold the cuff and wrap it around the upper arm evenly to the appropriate tightness.

The correct cuff position is shown in figure 2.5.

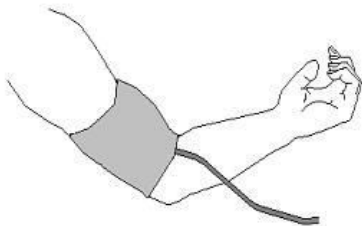



Figure 2.5 Cuff position

2. Connect the hose from the cuff to the connector on the upper-side of the device where marked “NIBP”.
3. Press the start/cancel button "  " to begin the blood pressure measurement.

#### **Safety Instructions for blood pressure measurement**

- Blood pressure measurement is prohibited to those who have severe hemorrhagic tendencies or with sickle cell disease, as partial bleeding may be caused.

- ☐ An appropriate cuff should be selected according to the age and arm circumference of the patient. The cuff width should be  $\frac{2}{3}$  of the length of the upper arm. The inflatable part should be long enough to permit wrapping approximately 80% of the limb. See the table below for the dimensions:

Cuff Model	Arm Circumference
Small-sized Adult Cuff	22cm~30cm
Middle-sized Adult Cuff	30cm~42cm
Large-sized Adult Cuff	42cm~48cm

- ☐ Continuous measurements may result in purpura, neuralgia and lack of blood.
- ☐ Do NOT wrap the cuff on limbs with transfusion tubes, intubations or skin lesions on the area, as damage may be caused to the limbs.
- ☐ The equipment can be used on pregnant or pre-eclamptic patients, but should not be used on neonatal patients.
- ☐ The patient should be as relaxed as possible and should not talk during the measurement procedure.
- ☐ Allow 5 minutes before taking a secondary reading.
- ☐ Readings can be affected by the measurement site, the position of the patient, and the patient's

physiological condition.

- ☐ The performance of the equipment can be affected by extreme temperature, humidity and altitude.
- ☐ Avoid compressing or restricting the connection tubing.
- ☐ The Patient should be comfortably seated with their legs uncrossed and feet flat on the floor. The patient's back and arm should be supported, and the middle of the cuff should be level with the right atrium of the heart.

Measurements should be taken at appropriate intervals. Continuous measurements with short intervals may lead to pressed arm, reduced blood flow low blood pressure, and result in an inaccurate reading. It is recommended that the measurements are taken in intervals of more than two minutes.

Before use, empty the cuff until there is no residual air inside. Do NOT allow the cuff to twist or bend.

Do NOT twist the cuff hose or put heavy things on it.

Please hold the connector of the hose while connecting and disconnecting it to the device.

If arrhythmia or auricular fibrillation occurs, take the measurement again.

### **2.3.2 SpO<sub>2</sub> Measurement**

#### **Operation procedures:**

1. Connect the smart SpO<sub>2</sub> probe to the connector on the upper-side of the device marked "SpO<sub>2</sub>" ("PORT1" or "PORT2" for previous versions of the device). When disconnecting the connector, be sure to hold the head of the connector firmly and pull.
2. The red blinking light inside the clip of the SpO<sub>2</sub> probe indicates a successful connection.
3. Insert one finger (index finger is preferred, the nail should be not too long) into the clip of the probe according to the finger mark, shown as below.
4. The device will begin to take the measurement.

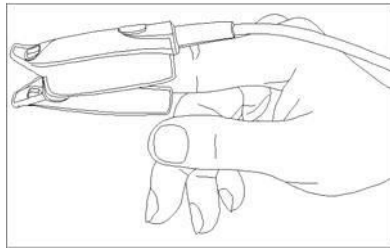


Figure 2.6 demonstration for SpO<sub>2</sub> probe

### **Safety instructions for SpO<sub>2</sub> measurements**

- ☐ Continuous use of the SpO<sub>2</sub> probe may result in discomfort or pain, especially for those with microcirculatory problems. It is recommended that the probe should NOT be applied to the same place for over two hours, change the measurement site periodically and when necessary.
- ☐ When the ambient temperature is over 35 °C, please change the measuring site every two hours; when the ambient temperature is over 37 °C, please do NOT use the SpO<sub>2</sub> sensor, as using in high temperatures can cause burns.
- ☐ Do NOT place the SpO<sub>2</sub> probe on a finger with edema or fragile tissue.
- ☐ Do NOT put the SpO<sub>2</sub> probe and pressure cuff on the same limb, otherwise the blood pressure measurement may affect the SpO<sub>2</sub> measurement.
- ☐ The device is calibrated to display functional oxygen saturation.
  - Do NOT allow the sensor cable to twist or bend.
  - Check the SpO<sub>2</sub> sensor and cable before use. Do NOT use a damaged SpO<sub>2</sub> sensor.
  - When the temperature of the SpO<sub>2</sub> sensor is abnormal, do not use it further.
  - Remove nail polish or other cosmetic products from the fingernail.
  - The fingernail should be of normal length.
  - The SpO<sub>2</sub> sensor cannot be immersed into water, liquid or cleanser.
  - The SpO<sub>2</sub> sensor can be repeatedly used. Please clean and disinfect before reuse.
- ☞ **Connectors with the label "SpO<sub>2</sub>"** can only be connected with the smart SpO<sub>2</sub> probe.

Note: The ECG and SpO<sub>2</sub> functions cannot be used simultaneously. If the device is successfully connected to both the ECG accessory and the smart SpO<sub>2</sub> probe, one function will take precedence over the other, for example if the user presses “Start” on the ECG accessory, the SpO<sub>2</sub> probe will be temporarily disabled until the ECG measurement is terminated.



### 2.3.3 Temperature Measurement

The infrared temperature probe is a delicate transducer. To operate please follow these steps and procedures. Failure to accurately operate may cause damage to the probes.

#### 1. The infrared temperature probe

Please place the infrared temperature probe in a stable ambient temperature for 30 minutes before taking a measurement.

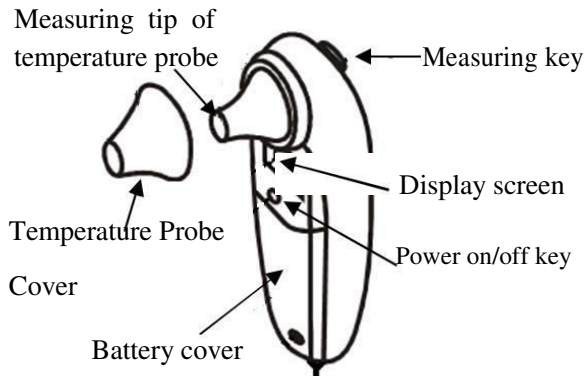


Figure 2.7A the infrared temperature probe

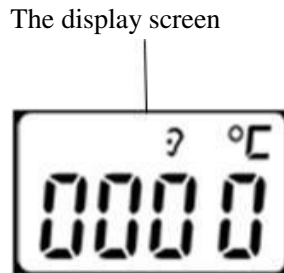



Figure 2.7B

**Operation procedure:**

1. Connect the infrared temperature probe to the connector on the upper side of device marked "TEMP"

("PORT1" or "PORT2" for previous versions of the device). When the LCD screen displays "", this indicates that the probe is successfully connected.

2. When the screen shows as figure 2.7B and the temperature unit "°C" is blinking, the user can begin to take the measurement.

3. Insert the tip of the temperature probe into the earhole and press the measuring key to start the measurement. A short beep means the measurement has finished and the result will be displayed on the screen.

**Note:**

¾ If the temperature probe detects a hardware failure, the display screen on the infrared temperature probe will show "Err" and will not enter into measuring mode.

¾ The infrared temperature probe will switch to stand by automatically if there is no operation for 1 minute. If a further measurement is needed, press the measuring key and repeat step 2 and step 3.

Normal body temperature varies depending on the position/area the measurement is taken from. The following table shows the varying temperature = ranges of the different body positions.

**Temperature varying range at different body positions:**

Arm	34.7 ~ 37.3 °C
Oral	35.5 ~ 37.5 °C
Rectal	36.6 ~ 38.0 °C
Ear	35.8 ~ 38.0 °C



### Safety Instruction for Temperature Measurement

Do NOT take a measurement when the patient is moving.

Patients with tympanitis and otitis problems should NOT use this device.

When the infrared temperature probe is connected to the device, the probe will consecutively be at power-on status, therefore pressing the power on/off key on the temperature probe will not cause any effect.

### 2.3.4 Blood Glucose Measurement (Optional)

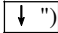Using the optional link cable for the **Super Check** Glucose Meter, connect the **Super Check** Glucose Meter to the connector on the right side of the Spot-Check Monitor marked "GLU" (" ↑  "). When the device shows " - -", this indicates that the blood glucose meter is linked successfully.

#### Appearance and key functions of the Super Check Glucose Meter:

1. Test strip slot: when the strip is inserted into the slot, the meter will automatically turn on.

- LCD display.
- M key: power on/off, also for memory recalling mode.
- C key: Setting mode. Please refer to User Guide for "Super Check Blood Glucose Monitoring System" for detailed function descriptions.
- Data interface: can be used to connect the Spot-Check Monitor for data transmitting.
- Battery compartment: insert 2 AAA size batteries with the correct polarities.
- Ejector: remove the used strip.

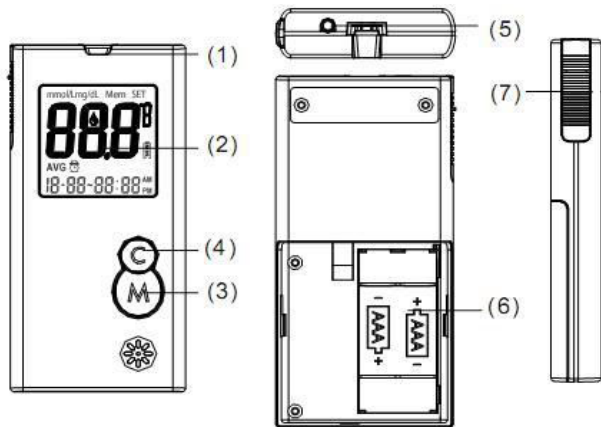


Figure 2.8 Appearance and functions of the Super Check Glucose Meter

### Operations for the Lancing Device and Blood Lancet

- Unscrew the lancing device by turning the end cap counter clockwise.
- Insert a new lancet firmly into the lancet holder.
- Twist off the protective tip of the lancet.
- Close the end cap of the lancing device. Slide into the locking position.

Refer to figure 2.9A

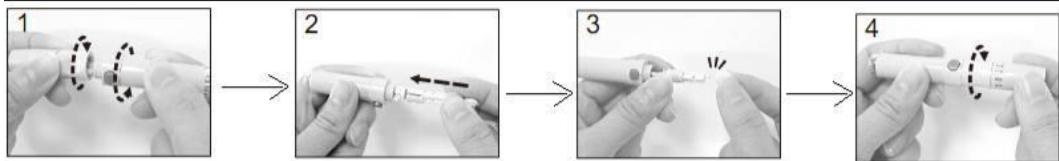


Figure 2.9A Operation for Lancing Device and Blood

**Lancet Quick operation procedure for Super Check Glucose Meter:**

1. While the meter is off, insert a new test strip into the meter. The meter will automatically turn on and a blinking blood icon will be displayed on screen.

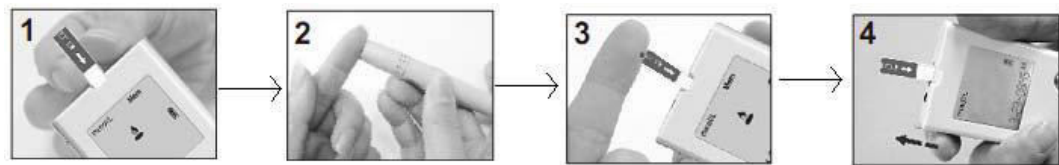


Figure 2.9B Testing instruction

2. Lance the finger and let a blood drop form.
3. When the blood drop icon is still blinking on the meter, apply the blood drop to the front edge of the test strip. The Meter will display the test result after 6 seconds
4. Remove the used strips by hand or by pushing the ejector and the meter will turn off and display "OFF" on the screen.

**Refer to the provided user guide for the "Super Check Blood Glucose Monitoring System" for further detailed instructions.**

### **Safety Instructions for Blood Glucose Measurement**

The provided test strips should be used with the **Super Check** Glucose Meter.

Do NOT clean or disinfect the finger with iodine.

The calibration code must be the same with that on the packaging.


The **Super Check** Glucose Meter will automatically switch to stand-by mode if a test strip is not inserted for 1 minute.

The test strip will draw blood at one end automatically.

Do NOT press or scrape the bleeding finger.

The test strip should be used as soon as possible after unpacking, and the unused strips should be kept in an airproof bottle.

Only take one measurement per minute.

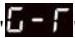☞ If the monitor is connected with both the temperature probe and the blood glucose meter, the screen will show "".

☞ The blood collection pinhead is a disposable item. It is recommended to insert it back into the plastic cover and throw it into a specific dustbin.

### **2.3.5 ECG Measurement (Optional)**

1. Connect the ECG accessory to the connector on the upper side of device marked "ECG" ("PORT1" or

"PORT2" for previous versions of the device).

2. Choose one of the methods (refer to figure 2.10B/C/D/E) to take the ECG measurement.
3. When the ECG accessory is connected to Spot-Check Monitor successfully, press the "Start" button on the ECG accessory to activate the ECG measurement.
4. When "ECG" appears on the display screen of the Spot-Check Monitor, it means the ECG accessory has begun to take the ECG measurement.
5. 30 Seconds later, the result will display on the screen of the terminal device, and the measurement will terminate.

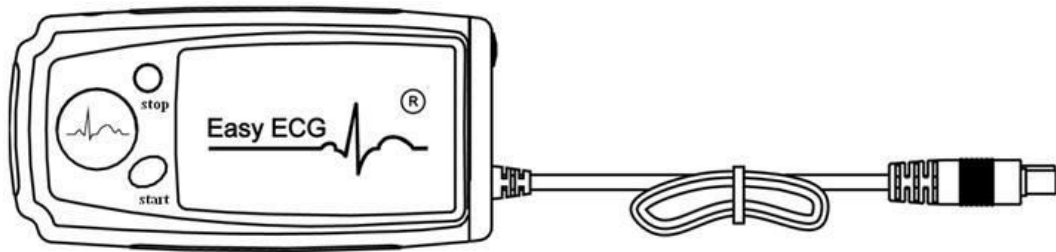


Figure 2.10A ECG accessory

¾ Start / Stop: Start/Stop ECG measurement.



Figure 2.10B

Palm measurement

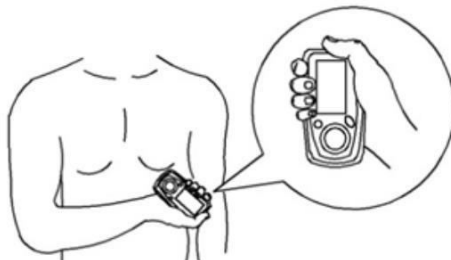


Figure 2.10C

Chest measurement



Figure 2.10D

Leg measurement

To obtain a clear and high quality ECG signal, the lead wire measurement can be used. Connect the lead wire firmly to the lead wire socket of the device. Place the electrodes and connect the lead wires as shown in Figure 2.10E to obtain the Lead II ECG signal. If you want to measure Lead I and Lead III ECG signal, connect the lead wires to the electrodes (note: lead wire is optional) as detailed in table below.

#### ¼ Safety Instructions for ECG Measurement

1. Check the device to make sure that there is no visible damage that may affect the user's safety and the measurement performance. If there is obvious damage, stop using the unit.
2. Do NOT make a diagnosis of oneself according to the measurement results, always consult a doctor if abnormal

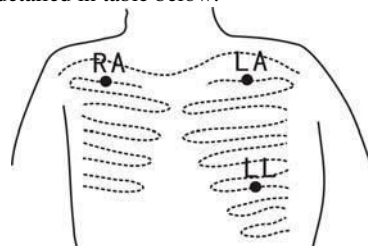


Figure 2.10E Lead wire measurement



information is presented frequently.

3. Do NOT use the device in a bathroom or humid environments.

**Table 1 ECG Leads Configuration and Electrodes Location**

<b>Electrode Name &amp; Color</b> <b>Electrode Location</b>	<b>Lead I</b>	<b>Lead II</b>	<b>Lead III</b>
The intersection between the centerline of the right clavicle and Rib 2.	R (Red)/ RA(White)	R (Red)/ RA(White)	L (Yellow)/ LA(Black)
The intersection between the centerline of the left clavicle and Rib 2.	F (Green)/ LL(Black)	L (Yellow)/ LA(Black)	R (Red)/ RA(White)
Between the left edge of the breast bone and Rib 5	L(Yellow)/ LA(Black)	F (Green)/ LL(Black)	F (Green)/ LL(Black)

## 2.4 Blood Pressure Accuracy Check

### Method Operation procedure:

1. Unscrew the M3x6 screw from the battery compartment on the back of the Spot-Check Monitor, as shown in figure 2.11.
2. Take a NIBP connector plug from the battery cover, as shown in figure 2.12. (Note: there are two plugs but you will only need one.)
3. Air Path Connection: Take a piece of air tube (0.5~1m long,  $\Phi 8.0\text{mm}/\Phi 4.0\text{mm}$  diameter). Attach the NIBP connector with a connector plug on to one end of the air tube. Connect the other end to the 3-way connector. Connect the other 2 ends of the 3-way connector to a Mercury Sphygmomanometer as shown in Figure 2.13.
4. Connect the NIBP connector end to the NIBP port on the Spot-Check Monitor as shown in figure 2.14.
5. Turn on the Spot-Check Monitor. Press the menu button to go to the settings. Press and hold the large NIBP measurement button to enter the Pressure Check Mode.
6. Start pumping, and check if the pressure reading on the Spot-Check Monitor matches the mercury pressure reading.



Figure 2.11



Figure 2.12

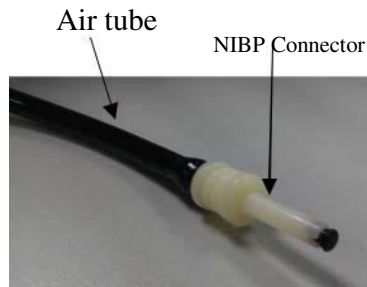


Figure 2.13















Figure 2.14








Figure 2.15

## 2.5 Symbols

Symbol	Description	Symbol	Description
	Wireless		Pulse rate (unit: bpm, beat per min)
	Alarm		Battery voltage indicator
	Memory icon		USB icon
	Pulse strength bar graph	bpm	Unit of pulse rate
°C/°F	Unit of temperature	kPa/mmHg	Unit of blood pressure
	Power on/off switch	DC 5.0V 1.2A 	External DC power input
	Type BF applied parts		Charger or USB data interface
SN	Serial Number		Connector to link with blood glucose meter

## User Manual for Spot-Check Monitor

	Refer to manual		Follow WEE regulations for disposal
TEMP	Connector for temperature probe	SpO <sub>2</sub>	Connector to smart SpO <sub>2</sub> probe
NIBP	Connector for cuff		Battery cover
ECG	Connector for ECG accessory		
	Manufacturer information		No Alarms

## CHAPTER 3 MONITORING SCREEN DISPLAY




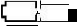



### 3.1 Measuring Screen



Figure 3.1B Measuring screen

#### Screen Description:

1. : USB connection icon
2. : wireless transmission icon; "": means that the wireless transmission function is on; when the icon is blinking, the wireless connection set up is unsuccessful; when this icon is steady, the wireless connection set up is successful; "": the wireless transmission function is off.

3. : Beep sound indicator; : pulse beep is on; : pulse beep is off.
4. : Battery voltage indicator. When the battery is full, the battery voltage indicator displays a full grid. When the indicator is blinking, it means the battery voltage is low and the user should charge the battery. Please connect the device to the external power supply to ensure the correct use of the monitor, and the battery will be charged. During charging, the grids in the battery indicator will roll circularly.
- 5~10. : Means the inflation pressure during cuff inflation. When displaying the measurement result, the description for the pressure will be displayed, such as O (Optimal), N (Normal), H (High normal), G1 (Grade 1 hypertension), G2 (Grade 2 hypertension), and G3 (Grade 3 hypertension).
11. M: Memory
12. ID: the patient ID, which can be set from 0 to 99.
13. NO.: the number of stored data, up to 999 records can be stored for each ID.
14. H:M: the time stamp (hour:minute). The time can be set in the system setup screen.
15. M-D: the time stamp (month-day). The date can be set in the system setup screen.
16. SYS: Systolic pressure
17. DIAS: Diastolic pressure
18. kPa/mmHg: unit of blood pressure, 1kPa=7.5mmHg.
19. SpO<sub>2</sub>: the value of SpO<sub>2</sub> with unit of %.
20. : Pulse bar-graph.
21. PR: Pulse rate with unit of bpm.
22. : the heart beat symbol, which flashes with heart beat.

23. TEMP/BG: the current displayed temperature with an option of °C for Celsius, or°F for Fahrenheit.

When the optional BG is chosen, the blood glucose value will be displayed with the default unit of mmol/L.

### 3.2 System Setting Screen










On the measurement display screen, press and hold the menu key to setup the display screen, as shown in figure 3.2. The user can choose the settings for the wireless function, pulse beep, blood pressure unit, temperature unit, date and time




Figure 3.2 Setup display screen



### Operation Instructions:

1. Press and hold the "" key, and release after hearing one beep, to enter into the setup screen. When the patient ID blinks, the setup function is available.
2. A short press of the "/ " key enables or disables the wireless transmission function.
3. A short press of the "" key confirms the setting. The beeping mark "" will blink.
4. A short press of the "/ " key enables or disables the pulse beep.
5. A short press of the "" key confirms the setting. The "kPa" (blood pressure unit) will blink.  
The functions of wireless transmission, beep, blood pressure unit, temperature unit, date and time can be set by following the above steps.
6. Press and hold the "" key to bring the screen display back to the measurement display screen. The monitor will also switch back to the measurement display screen if there has been no operation for 30 seconds.

Note: 1. On the setup display screen, all parameters can be set in anticlockwise order by pressing and holding the "" key.



135  
11:14  
03-23

2. For setting the date, the century is fixed to be 20, i.e. "13y" indicates the year 2013. Please see the following example for the date and time: 11:14", March 23, 2013.

### 3.3 History Data Review Screen

On the measurement display screen, press and hold the "⏪" key to recall the stored data records, as shown in figure 3.3

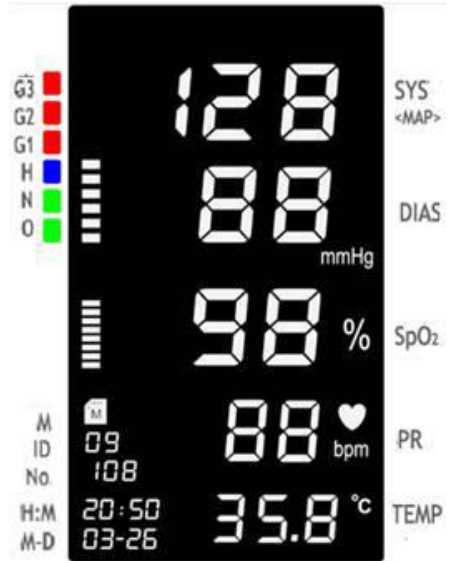









Figure 3.3 History data review screen

### Operation instructions:

1. Press and hold the " " key, release the key after hearing one beep. The memory mark " " will appear (i.e. entering to review display screen). The patient's ID number will blink at the same time.
2. A short press of the " /  " key will browse the patient's ID numbers.
3. A short press of the " " key will confirm the setting, and the recorded number (No.) will blink.
4. A short press of the " /  " key will set the recorded number to be recalled. The data displayed on the screen is for the specific record of the patient selected.

Note: when selecting the patient ID, the screen only displays patients with history data records.

### 3.4 Data Uploading

1. When the wireless transmission function is on, the monitor can communicate with a host device such as a PC, smart phone or other wireless enabled devices for real-time data transmissions.
2. When connected to a USB cable, the history data can be uploaded to a PC for viewing and management.

## TECHNICAL SPECIFICATIONS

### 1 Blood Pressure Measurement

1. Technique: Oscillometric
2. Pressure measuring range: 0mmHg~300mmHg
3. Cuff inflation time:<20 seconds (typical adult cuff)
4. Accuracy of pressure measurement:  $\pm 3\text{mmHg}$
5. Overpressure protection limit  
Adult:  $\leq 300\text{mmHg}(39.9\text{kPa})$
6. Blood pressure measurement range:  
SYS: 60mmHg~240mmHg      DIA: 30mmHg~180mmHg
7. Blood pressure measurement accuracy:  
Maximal mean difference:  $\pm 5\text{mmHg}$   
Maximal standard deviation: 8mmHg

### 2 SpO<sub>2</sub> Measurement

1. Technique: optical with dual-wavelength

Monitor LED wavelength: Red light: 663nm, Infrared light: 890nm

Maximal optical output power: less than 2mW maximum average

2. SpO<sub>2</sub> displayed range: 35%~100%
3. SpO<sub>2</sub> measuring accuracy: A<sub>rms</sub> is not greater than 3% for SpO<sub>2</sub> range from 70% to 100%  
Note: A<sub>rms</sub> is defined as root-mean-square value of deviation according to ISO 9919
4. Low perfusion performance: the above declaration is still attained while the amplitude modulation ratio is as low as 0.6%.

### **3 Pulse Rate Measurement**

1. PR measuring range: 30bpm~240bpm
2. Pulse rate measuring accuracy:  $\pm 2$ bpm or  $\pm 2\%$ , which is greater

### **4 Temperature Measurement**

1. Measuring range: 32.0°C~43.0°C
2. Measuring accuracy:  $\pm 0.2$ °C is for TEMP range from 35.0°C to 42.0°C, and  $\pm 0.3$ °C is for the rest.
3. Response time:  $\leq 5$ s

## 5 Blood Glucose Measurement (Optional)

1. Technique: Amperometric, glucose oxidase
2. Measuring range: 1.1mmol/L~33.3mmol/L (20~600mg/dL)
3. Measuring time: 6 seconds

## 6 ECG Measurement (Optional)

1. Heart Rate measuring range: 30bpm~240bpm
2. Heart Rate measuring accuracy:  $\pm 2$ bpm or  $\pm 2\%$  whichever is greater
3. Display scale: 5.0mm/mV $\pm 10\%$
4. Common-mode rejection ratio (CMRR):  $\geq 60$ dB

## 7 Others

### 4.7.1 Operating Environment

1. Operating temperature: 5°C ~ 40°C;                      Relative humidity: 15% ~ 93%;  
Atmospheric pressure: 70.0kPa ~ 106.0kPa;              Power supply: 110V-240VAC, 50/60Hz, 15VA;
2. The device should be situated in a place protected against direct sunlight, to prevent overheating inside

of the equipment.

3. Do not use this equipment in combination with any equipment other than those approved in the user manual.
4. The device should be stored and used in a specified temperature, humidity and atmospheric pressure range, or damage may be caused to the device and as a consequence, record inaccurate results.
5. If the device gets wet by accident, the operator should NOT turn on the power until it has been thoroughly air dried.
6. Do not use this equipment in an environment with toxic or inflammable gases.
7. Monitor with sizes 4.3 inch and 9 inch.

Warning: Do not use any other adapters than those provided by Creative.

#### **4.7.2 Classification**

1. Protection against electric shock: Class II equipment and internally powered equipment
2. The degree of protection against electric shock: Type BF applied part
3. Define apply part: cuff, SpO<sub>2</sub> probe, temperature probe, ECG lead wires (optional).
4. The degree of protection against harmful ingress of liquid: The equipment is IP22 with protection against ingress of liquid
5. Electro-magnetic Compatibility: Group I, Class A

## CHAPTER 5 TROUBLESHOOTING

Trouble	Possible reason	Solution
Cannot turn on the device	The built-in battery is drained	Recharge by connecting the power supply adapter
	Battery is not installed	Install the Lithium battery
	Some parts provided by other manufacturers are inserted to the connector	Remove the related parts and try again.
No blood pressure result	The cuff is wrapped around the arm incorrectly	Wrap the cuff around the arm correctly
	The windpipe is not correctly inserted to the NIBP jack	Insert the windpipe to the NIBP jack
No SpO <sub>2</sub> result	The SpO <sub>2</sub> probe is not plugged into the "SpO <sub>2</sub> " connector (or Port1 or Port2)	Plug the SpO <sub>2</sub> probe into the "SpO <sub>2</sub> " connector (or Port1 or Port2)
No TEMP result	The temperature probe is not correctly plugged into "TEMP" connector (or Port1 or Port2)	Plug the temperature probe into "TEMP" connector (or Port1 or Port2)
	Taking measurements before "READY" appears on the temperature probe screen	Do not take a measurement until "READY" appears on the temperature probe screen



## CHAPTER 6 PACKING LIST

Item	Description	Quantity	Check
1	Spot-Check Monitor	One piece	OK
2	Handbag	One piece	OK
3	User Manual	One piece	OK
4	Cuff	One piece	OK
5	USB cable	One piece	OK
6	Charger (with USB socket)	One piece	Optional
7	Temperature probe	One piece	
8	Smart SpO <sub>2</sub> probe	One piece	
9	<b>Super Check</b> Glucose Meter (with lancing device and link cable )	One set	
10	Blood glucose test strips (with blood lances)	One pack	
11	Spot-Check Monitor Data Manager Software	One set	
12	ECG accessory	One piece	
13	ECG lead wire (snap)	One piece	
14	Disposable adhesive ECG electrodes	Six pieces	

## **CHAPTER 7 MAINTENANCE AND SERVICE**

The Spot-Check Monitor should be properly maintained to ensure its maximum performance and long service life. In addition to the warranty period, the company also offers long-term service for each customer. It is important that the user reads and follows the operation instructions, important information and maintenance measures.

### **7.1 Technical Maintenance**

#### **7.1.1 Daily Examination**

Before using the monitor, the following checks should be carried out:

- „Check the monitor for any mechanical damage;
- „Inspect the exposed parts and the inserted parts of all the leads, and the accessories;
- „Examine all the functions of the monitor that are likely to be used for patient monitoring, and ensure that it is in good working condition.

If there is any indication of damage, or if damage is accurately proven, do not use the device. Contact your supplier for advice and to reach a satisfaction solution.

#### **7.1.2 Routine Maintenance**

The PC-303 Spot-Check monitor is designed to have a life expectancy of at least 5 years. In order to ensure precise readings, it is recommended to test the pressure accuracy every year. Please see section 2.4 to

perform the Blood Pressure Accuracy Check Method or contact your supplier for more information. After each maintenance or yearly maintenance, the monitor should be thoroughly inspected by a qualified personnel, including an inventory of all functions and safety examinations.

- If the hospital fails to carry out a satisfactory maintenance program on the monitor, it may cause harm to the patient.
- If there is any indication of cable and transducer deterioration or damage, please do not use.
- The SpO<sub>2</sub> function has been adjusted before vending. If the user needs to adjust the SpO<sub>2</sub>, adjust by using the simulator mode FLUKE INDEX2.

The adjustable units in the monitor such as the potentiometer cannot be adjusted without permission so as to avoid unnecessary failures that may affect the normal application.

It is recommended to use the battery once a month to ensure a strong power supply capacity and long service life, and recharge once the power has completely run out.

### **7.1.3 Battery Maintenance**

- Please pay attention to the polarity of the battery, do NOT insert into the battery compartment with reversed polarities.
- In order to avoid damaging the battery, do NOT use other power supply devices to charge the battery.
- After use, dispose of the battery according to local regulations, do NOT throw into fire.

- ❑ Do NOT hit or strike the battery with force.
- ❑ Do NOT use this battery in other devices.
- ❑ Do NOT use this battery below  $-20^{\circ}\text{C}$  or above  $60^{\circ}\text{C}$ .

In order to maintain the battery supply and prolong the battery lifetime, please charge the battery routinely. Regularly, charge the battery every 3 months even if the device has not been used.

Only use a battery with the specification recommended by the manufacturer.

Whether the monitor is on or off, the built-in battery will charge as long as the monitor is connected to an AC adapter and the AC power is on. When the battery is full, it will stop charging to avoid causing any damage. If the monitor is connected to an AC adapter and the AC power is on, it will use the AC power, but when the AC power is off, the battery power will be used. Priority of using the AC power and the power switch between AC and battery is automatic and seamless.

If the battery is damaged, please replace it with a battery with "CCC" or "CE" mark. The model and specifications of the battery should be the same as the original battery. The user must ensure that the battery meets all applicable safety codes. The user can also contact the distributor for service.

## **7.2 Cleaning and Disinfection of the Main Unit**

Switch off the monitor and disconnect the power cord before cleaning.

Keep the monitor free from dust.

It is recommended to regularly clean the outer shell and screen of the monitor. Only use a

non-corrosive cleanser such as clear water.

Wipe the surface of the monitor and transducers with an alcohol impregnated wipe, and dry with a clean cloth or just air-dry.

Dilute the cleaner.

Do NOT use scrubbing materials.

The monitor can be disinfected. To avoid damage do not let liquid cleaner flow into the connector jack of the monitor.

Clean the exterior of the connector only.

Do NOT let any liquid flow into the shell or any other parts of the monitor.

Do NOT leave any residue liquid or disinfectant on the surface of the monitor.

Do NOT perform high pressure sterilization on the monitor.

Do NOT immerse any parts of the monitor or its accessories in liquid.

If the monitor accidentally becomes wet, it should be thoroughly dried before use. The rear cover can be removed by a qualified service technician to verify the absence of water.

Do NOT pour disinfectant on the monitor's surface while disinfecting.

### **7.3 Cleaning and Disinfection of Accessories**

It is recommended to clean and disinfect the accessories (excluding the SpO<sub>2</sub> probe) with a piece of gauze soaked in 75% Alcohol or 70% Isopropanol.

Do not use damaged accessories.

Accessories cannot be entirely immersed into water, liquid or cleanser.

Do NOT use radiation, steam or epoxyethane to disinfect accessories.

Wipe off any remaining residue of alcohol or isopropanol after disinfection.

Disinfect the temperature sensitive probe with alcohol.

Wipe the thermometer clean with a mild cloth if it becomes dirty.

Wipe the thermometer clean and return to packaging after use.

## 7.4 Storage

If the equipment will not be used for a long time period of time, wipe it clean and return it to the packaging.

Store in a dry well ventilated place free from dust and corrosive gases.

Storage environment: Ambient temperature:  $-20^{\circ}\text{C}\sim 60^{\circ}\text{C}$

Relative humidity:  $\leq 93\%$

Atmospheric pressure:  $53.0\text{kPa}\sim 106.0\text{kPa}$

## 7.5 Transportation

The monitor should be transported by land (vehicle or railway) or air in accordance with the contractual terms. Do NOT hit or drop with force.

## Quality Certificate

Name: Spot-Check Monitor

Model: \_\_\_\_\_

Date: \_\_\_\_\_

QA: \_\_\_\_\_

This product has been inspected in accordance with  
the standards specified in the User Manual  
Shenzhen Creative Industry Co., Ltd.

## Warranty Clause

1. This monitor has a warranty of 12 months (including rechargeable battery) and 6-month for all accessories, from the date of purchase.
2. It is recommended to use the original packing boxes and packing materials when returning for repair or maintenance
3. Please send the device to the specified place for repair.
4. The following will invalidate the warranty:
  - If the monitor is damaged due to misuse or incorrect operation (i.e. without the user manual instruction. –The monitor is damaged due to incorrect connection with another instrument –The monitor is accidently damaged, dropped or immersed into water
  - If the user modifies or changes the monitor without written authority of the company –If the serial number is deliberately damaged, torn off or unreadable
5. If the monitor is non-functional outside of the warranty period, the manufacturer or distributor will offer an estimate for repair.



## Warranty

<b>Device Information:</b>			
Name		Model	
Serial Number:			
Date		Shop	
<b>User Information:</b>			
Name		Postcode	
Tel:			
Add:			
<b>Repair Record</b>			
Date	Repairing Item		Personnel

## **Patent**

State Intellectual Property Office of the P.R.C. patented and certificated Creative Spot-Check Monitor on March 26<sup>th</sup>, 2014. Patent Number: ZL 2013 2 0615696.X

## **Shenzhen Creative Industry Co., Ltd.**

**2/F, Block 3, Nanyou Tian'an Industry Town,**

**518054 Shenzhen, GD, P.R. China**

**Tel: +86-755-2643 3514**

**Fax: +86-755-2643 0930**

**E-mail: [info@creative-sz.com](mailto:info@creative-sz.com)**

**Website: [www.creative-sz.com](http://www.creative-sz.com)**

**EC-Representative: Shanghai International Holding Corp. GmbH (Europe)**

**Address: Eiffestrasse 80, 20537 Hamburg Germany**

